



AED Maintenance Checklist

15 Feb 2021 / SafetyCulture Staff

Complete

Score	84.62%	Failed items	2	Actions	3
Checked on	15th Feb, 2021 7:51 AM PST				
Checked by	SafetyCulture Staff				

Failed Items & Actions

2 failed, 3 actions

Failed items

2 failed, 3 actions

AED Maintenance Checklist / AED Maintenance Log

Are all cables free of cracks, cuts and exposed or broken wires?

Fail



Photo 3

To do | Priority Medium | Due 15th Feb, 2021 9:00 AM PST | Created by SafetyCulture Staff

Call AED technician to fix.

AED Maintenance Checklist / AED Maintenance Log

Batteries within expiration date. Replace if expired.

Fail



Photo 5

In progress | Priority High | Due 15th Feb, 2021 8:30 AM PST | Created by SafetyCulture Staff

Need to buy new AED batteries.

Can't do | Priority High | Due 15th Feb, 2021 11:00 AM PST | Created by SafetyCulture Staff

Replace ASAP.

Other actions

0 actions

AED Maintenance Checklist

3 actions, 2 failed, 84.62%

AED Maintenance Checklist

100%

- Inspect frequently, as necessary.
- Check for the green check showing that the AED is ready to use.
- Verify that electrodes are within their expiration date.
- Verify that batteries are within their expiration date.
- Verify that electrodes are pre-connected to the input connector.
- Verify that supplies are available for use (razor, mask, gloves, extra batteries.)

AED Maintenance Log

3 actions, 2 failed, 71.43%

Is the unit clean, undamaged, free of excessive wear?

Pass



Photo 1

Are there any cracks or loose parts in the housing?

Pass

Verify electrodes are connected to the AED and sealed in their package.
Replace if expired.

Pass



Photo 2

Are all cables free of cracks, cuts and exposed or broken wires?

Fail



Photo 3

To do | Priority Medium | Due 15th Feb, 2021 9:00 AM PST | Created by SafetyCulture Staff

Call AED technician to fix.

Turn the AED on and off and verify the green check indicates ready for use.

Pass



Photo 4

Batteries within expiration date. Replace if expired.

Fail



Photo 5

In progress | Priority High | Due 15th Feb, 2021 8:30 AM PST | Created by SafetyCulture Staff

Need to buy new AED batteries.

Can't do | Priority High | Due 15th Feb, 2021 11:00 AM PST | Created by SafetyCulture Staff

Replace ASAP.

Check for adequate supplies.

Pass



Photo 6

Appendix

Accessibility



Photo 1



Photo 2



Photo 3



Photo 4



Photo 5



Photo 6